

# B. Braun Sterican® Safety Needle

Single-use needle with safety protection



Carefully remove Sterican® Safety Needle from the packaging.



Attach the safety needle securely to any Luer-Lock or Luer-Slip syringe by twisting the syringe clockwise while holding the clear needle safety shield close to its base.




Hold the syringe with one hand and with the other hand fold the safety shield back towards the syringe.




Remove the protective cap carefully from the needle hub to avoid exposure to the sharp needle point.



Perform injection according to your established procedure.

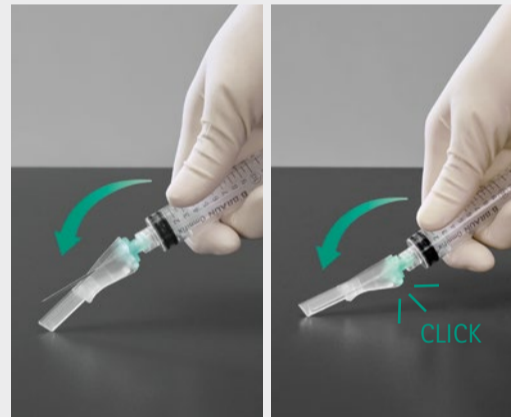

Activate the mechanism by pressing the safety shield against a suitable hard and flat surface until you hear an audible click.

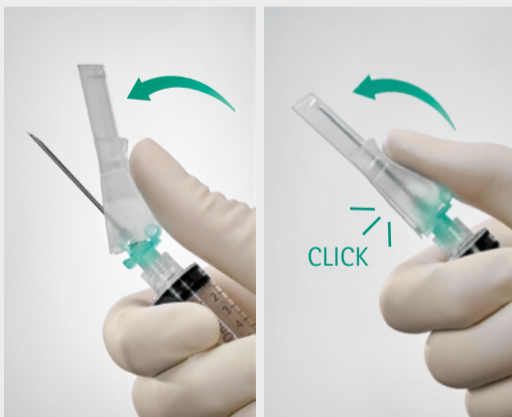

If activating with one hand, position your thumb or forefinger carefully on the textured finger plate. Push the safety shield over the needle until a slight snap is felt and an audible click is heard. Perform activation of the safety feature away from yourself and others.



Be sure to visually confirm that the safety feature is engaged before disposal.



After activation dispose the used needle immediately into an approved sharps container following the standard procedures of your facility and /or according to applicable regulations for sharps disposal.