DE

Dynamic Plus®

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Dynamic PLUS

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Dynamic Plus®

User Manuel

Resin-based dental restorative material

Description
 Dynamic is a visible-light activated, radiopaque, hybrid composite material. It is designed for use in both anterior and posterior restorations.

Composition
 - Barium aluminosilicate (Average Particle Size: 51µm)
 - Fumed silica (Average Particle Size: 0.04µm)
 - Bis-GMA
 - Triethyleneglycol dimethacrylate
 - Others
 - The percentage by weight of total inorganic filler is ca.80%

Indications
 Dynamic is indicated for class I, II, V of posterior teeth, class III, IV, V of anterior teeth and cervical cavities(wedge shaped defect) involving root surfaces.

Contra-Indications
 Dynamic should not be used with patients who develop hypersensitivity to it or have a history of hypersensitivity to methacrylate monomer.

Recommended Procedure

1. Shade selection
 Clean teeth with pumice and water to remove surface stains or extraneous plaque. Teeth are not monochromatic. Consider shade and restoration depth. It is desirable to choose the shade after mocking up. Alternatively, Vita 1 Lumin® Vacuum shade guide may be used. Using of rubber dam is recom mended for isolation.

2. Cavity preparation
 - Anterior restoration
 Using the conventional acid etching, prepare cavity for all Class III, Class IV, and Class V restoration.
 - Posterior restoration
 Prepare the cavity. No residual amalgam or other base material should be left on the internal surfaces of preparation that would interfere with light transmission and the hardening of the restorative material.

3. Pulp Protection
 In deep cavities cover the dentin close to the pulp with a minimum amount of calcium hydroxide liner leaving the rest of cavity surface free for bonding. Glass Ionomer or other Eugenol-free base materials may be used, if wished.

4. Placement of matrix
 Use a matrix system, preferably a transparent one, with proper wedging for proximal contacts. Pre-wedging is advocated to achieve slight separation and facilitate optimal proximal contact.

5. Enamel and dentin treatment
 Follow the manufacturer's instructions regarding etching, prim ing, adhesive application and curing. It is recommended to use Etching Gel and Prebond or Prebond SE

directly regarding the same quality matters.
 11. Do not use after expiry date.

Shade	Hickness	Exposure Time	
		1.000 m W/cm ²	400m W/cm ²
A1, A2, A3, A3.5, B1, B2, B3, C2, C3, I, P, E	2.5mm	10 san	20 san
A4, D3, A20, A3	2.5mm	15 san	30 san

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9. Finition
 Immédiatement après curing contour restoration surfaces with fine finishing bur or stone. Carefully adjust occlusion by removing material with a ine polishing bur or stone.

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